

Numera EverThere[©]

If you'd prefer to watch a video,
search YouTube for Numera Everthere App.

- You will receive an email invitation
- Click on the, “Register” Button
- Then enter the appropriate information and create a password
- Once you have completed registration and accepted the terms & conditions, you be asked validate your device
- Validation is only required for the initial set-up of your device
- Follow the directions for removing and replacing the device in the charger
- When you log-in you will be to the EverThere[©] **Dashboard**
 - The device’s last lactation, signal strength, battery and the Event Activity level are on the screen
 - If you click on the “settings icon” you can select which events will appear on the log. Hovering over the “?” mark will provide a description of the event
 - You can also create a “Location Tail” that will display the number of prior locations showing where the device up to 15 locations
- Click “OK” to save the information
- In the **Settings** tab you can adjust the information
 - If there is a “lock” symbol, **USAMedAlert** has fixed that field and you cannot make any adjustment
- In the **User Profile**, you can view the account
- In the, “My Information” icon, click the “pencil” icon to make changes
- The email address cannot be changed as that is how you are identified

- Under the **Team** icon, you can upgrade to the EverThere® Family plan
- If the option is not available and you would like to add additional family members and/or care taker(s) call us @ 262.909.3636 for information!
- NOTE: additional family members and/or care taker(s) cannot make changes to the account information. If you'd like, any additional can make changes. Call us for details!
- In the **Subscription** area is where the added people to the account can select which events, they will receive notifications and will the notification be sent via email and/or text message. This feature may not be available. If, not, please call us.

You must accept the emailed invitation within 30-days.